

Instructions for use - Please read carefully!

Lysoform®

Disinfection of surfaces, equipment and medical devices

Concentrate

Active ingredient: Formaldehyde

for professional use

Composition

100 g solution contains: 7.5 g Formaldehyde, Sodium alkyl ether sulphate

Range of application

Destruction of micro-organisms causing diseases, decay, putrefaction and/or smells

- Disinfection of surfaces, equipments and laundry
- Control of epidemics

Contra-indications

Allergy to formaldehyde

Mode of application / Dosage

Use externally and diluted with water only!

Close container immediately after taking the necessary amount of **Lysoform®** concentrate.

Disinfection and cleaning of surfaces and equipment:

Wet-wipe surfaces and equipment with a 1% Lysoform® solution - leave to act for 4 hours.

Laundry disinfection:

Soak laundry in a 2% Lysoform® solution during 4 hours.

Virucidal* incl. HBV/HCV/HIV

Scrub disinfection in case of epidemics:

According to RKI-list (A, B) 5.0 % - 6 hours

Dosing example: 1 litre of water and 10 ml of Lysoform® = 1 % solution

Warnings / storage

- Keep disinfectant in a dry, cool but frost free and well ventilated place and out of reach of children!
- Wear suitable protective clothing, adequate protective gloves and eye protection when handling the concentrated product.
- Do not use beyond expiry date.

Properties

- Disinfection and cleaning of floors and objects contaminated by skin fungus (bath-mats, grids, shoes and socks etc.)
- Laundry and textile disinfection (tb incl.), disinfection at the hospital, at the doctor's surgeries and hygienic maintenance of all sanitary equipment, e.g. toilets and bath rooms
- Biodegradable

Special precautions

a) Counter-measures in case of improper use (e.g. throat, stomach etc.) If the **concentrated** preparation has come into contact with the mucous membranes – and also after contact with the eyes in this case even if **the product is diluted** - rinse **thoroughly** with plenty of water and get medical attention immediately.

If **Lysoform®** has been **swallowed**, make patient drink **plenty** of water to dilute the product and get medical attention immediately! Do not make patient vomit, because alkali burns might occur in the esophagus.

b) Adding the usual detergents and/or household cleaners to **Lysoform®** can impair or neutralize its disinfecting properties.

c) Use disinfectants safely. Always read the label information and instructions before use.

Translated by Lysoform Dr. Hans Rosemann GmbH, Berlin - Germany

use!

Standard packaging / transport units

20 x 500 ml bottle
12 x 1 litre bottle
2 x 5 litre canister

Article-No:

6545
6566
6444

Accessories

Dosing pump for 5 litre canister (1 pression = 20 ml)

Lysoform Dr. Hans Rosemann GmbH
Kaiser-Wilhelm-Str. 133, D – 12247 Berlin
Tel.: +49-30-77992-0 Telefax: +49-30-77992-219

0518-7