Instructions for use - Please read carefully!

# Lysoformin® Plus-Schaum

Ready for use

fast-acting disinfecting and cleaning foam for medical inventory

## For professional use

## Composition

100 g solution contains:

0,18 g Polyhexamethylene biguanide HCI,

0.46 g Bis-(3-aminopropyl) dodecylamine,

0.10 g Didecyl dimethyl ammonium chloride, fragrances

## Range of application

Fast-acting disinfectant for medical inventory within the meaning of the Medical Devices Act (MPG). Also suitable for other water-resistant surfaces.

- Disinfection and cleaning of surfaces (including medical devices) in a single operation
- · Application according to reprocessing instructions of the medical device manufacturer

## Mode of application / dosage

Lysoformin<sup>®</sup> Plus-Schaum is ready for use. Spray foam on the surfaces and objects to be disinfected and allow to act, then eliminate with a clean, damp cloth. Product can also be applied undiluted on a one-way cloth which will be used to wipe the surfaces to be disinfected and cleaned.

Contact time: 5 minutes

Virucidal against enveloped viruses\*

Incl. HBV/HCV/HIV 5 min Polyomavirus (SV 40) 15 min

#### Warnings / storage

- Keep disinfectants in a dry, cool but frost free and well ventilated place and out of reach of children.
- Do not inhale spray or foam, if necessary wear respiratory protection with a suitable particle filter.
- Wear adequate protective gloves and protective clothing when handling the product.
- Do not use beyond expiry date.

#### **Properties**

- control of bacteria, fungus, enveloped viruses
- optimal wetting action of the surfaces and thus gentle and safe disinfection
- · easy handling, ready to use
- · freshly scented

#### Special precautions

a) Counter-measures in case of improper use:

Should Lysoformin<sup>®</sup> Plus-Schaum come into contact with the mucous membranes (eyes, throat, stomach, etc.) through improper use get medical attention immediately. Dilute product immediately: if it has been swallowed, drink **plenty** of water; after contact with the eyes, rinse thoroughly with **plenty** of water.

c) Miscellaneous:

Rinse objects/surfaces that come in contact with food with drinking water after the recommended exposure time e.g. after completed disinfection.

d) Use disinfectants safely. Always read the label and product information before use.

#### Standard packaging / transport units article numbers

12 x 1 litre (bottle) 2505 2 x 5 litres (canister) 2509

Lysoform Dr. Hans Rosemann GmbH Kaiser-Wilhelm-Str. 133, D – 12247 Berlin

Tel.: +49-30-77992-0 Telefax: +49-30-77992-219

1115-5

<sup>\*</sup>according to recommendation of Robert Koch Institute Berlin/ Federal Health Gazette 01/2004